

WARNINGS & CAUTIONS

Discontinue treatment & consult your eye care professional if discomfort is noticed after the first week and persists for more than 10 minutes.

DO NOT re-use Soft Tips. Dispose after a single treatment to avoid infection or injury.

DO NOT share a NuLids Soft Tip with another person.

Keep the handpiece dry and free from excess liquids.

Refer to NuLids Warranty and the User Manual located at www.NuSightMedical.com for complete care and instructions for use.

DO NOT put the Soft Tip on your eye or anything other than the eyelids.

DO NOT conduct treatment more frequently or for a longer period than recommended.

AVAILABLE RESOURCES

Visit www.NuLids.com to:

- Read the User Manual
- Watch videos on “How to Use NuLids”
- Email or call our Customer Solutions team

NUSIGHT MEDICAL CUSTOMER SOLUTIONS (for US customers)

Email: CS@NuLids.com (or)

Phone: 1-833-368-5437, ext 1

- Get answers to product usage questions
- Order more NuLids Soft Tips
- Enroll in the automatic refill program
- Share your success story!



Scan me with your smartphone to register your warranty and watch tutorial videos!



NuLids[™]
at-home dry eye relief

Published: 2/20/2021

Revision: C



NuLids[®]

QUICK START GUIDE*



STEP 1: CHARGE THE HANDPIECE

First Time Use: Charge handpiece fully until solid green light seen.

Future Uses: Re-charge handpiece when amber light flashes. Charging is complete when the green light stops flashing or turns off.

STEP 3: PREPARE FOR USE

- Remove eye makeup and wash hands.
- Press On/Off button on handpiece once.
Result: White light on tip.
- Align opening on top of handpiece with peg on Soft Tip until it magnetically snaps into position. Confirm the Tip is firmly in place.



- Apply a gel or lubricant for extra comfort as follows:
 - **When using NuLids Revitalize Eyelid Gel**, or similar product, apply one drop to the front of the Soft Tip, or
 - **When using a foam, spray, or oil**, do not apply to the Soft Tip. Instead, apply directly on the closed eyelid and/or the eyelid margin.



STEP 4: BEGIN TREATMENT

- **Go to Step 4A (Indirect Method) or**
- **If recommended by your doctor, Go to Step 4B (Direct Method)**

STEP 4A: INDIRECT METHOD



- Close one eye gently.
- Place edge of Soft Tip at intersection of upper and lower lids at the lash line and eyelid margin.
- Press the “On” button.
- Move oscillating Soft Tip slowly with even pressure from one side of the lid to the other for 30 seconds.
- Move to opposite eye for 30 seconds and treat in same manner.
- Press On/Off button to stop handpiece.
- Dispose of used Soft Tip.

Note: Handpiece pauses after 15 seconds. Pauses are designed to give estimate on treatment length.

Suggestions

Use a mirror until you are comfortable with tip placement.

Be conscious of where you place the face of the Soft Tip prior to treatment.

Unplug handpiece from the USB cord prior to treatment.

Experiment with holding the handpiece to find what is comfortable.

Keep the Soft Tip in contact with the lash line and eyelid margin for best results.